

MPG Healthcare

Antigen Rapid Test

Antigen rapid test (colloidal gold) Nasopharyngeal



Immuno-graphic rapid in vitro test

SPECIFICATIONS

- Nasopharyngeal smear
- Rapid result (10 - 15 minutes)
- Easy to use
- One step test, all materials included in this kit
- For professional use

STORAGE CONDITIONS

In a dry well ventilated room (2°-30° C), application at 15° - 30° C.

PACKAGING UNITS AND CONTENT

Package sizes: 1 test set, 25 test sets

- Test strips (individually in a foil pouch with desiccant)
- Sterile swab
- Extraction buffer (buffer solution separate included)
- Extraction tube
- Instructions for use

APPLICATION

A smear is taken from the nasopharynx. The swab is taken from the front area of the nostrils (left and right). The sample is extracted by rotating the test swab in the sample tube and then squeezing the sample tube into the solution. After taking the sample aside for a short time, 2 drops of the solution can now be added to the test cassette. You will see the result in 15 minutes.

- Shelf life 05/2023

CHARACTERISTICS

The COVID-19 Antigen Rapid Test was compared to a commercial PCR.

Total sensitivity 96.1% ; 95% CI:(88.9% - 99.2%)
 Total specificity is 98.1% ; 95%CI:(95.3%-99.5%)
 Total compliance is 97.6% ; 95%CI:(95.1%-99.0%)

Please refer to the further information in the instructions for use.

Antigen rapid test (colloidal gold) Anterio nasal



Immuno-graphic rapid in vitro test

SPECIFICATIONS

- Anterio Nasal swab
- Easy to use
- Rapid result (15 minutes)
- Test intended for self use in everyday situation
- For professional use
- Approved for self-testing in Austria . Listed with BASG since February 2021

STORAGE CONDITIONS

In a dry well ventilated room (2°-30° C), application at 15° - 30° C.

PACKAGING UNITS AND CONTENT

Package size: 1 test set, 25 test sets included

- Test strips (individually in a foil pouch with desiccant)
- Sterile swab
- Extraction buffer (buffer solution sparate included)
- Extraction tube
- Instructions for use

APPLICATION

The swab is taken from the front area of the nostrils (left and right). The sample is extracted by rotating the test swab in the sample tube and then squeezing the sample tube into the solution. After taking the sample aside for a short time, 2 drops of the solution can now be added to the test cassette. You will see the result in 15 minutes.

- Shelf life 1 year (marked on the product)

CHARACTERISTICS

The COVID-19 Antigen Rapid Test Anterio Nasal was compared with the Novel Coronavirus Real Time Multitplex RT-PCR Kit.

The total sensitivity of COVID-19 Antigen is 96.1%: 95% CI: (90.26%~98.92%)

Total specificity of COVID -19 Antigen is 99,2%, 95% CI (97,96% - 99,78%)

Total agreement rate of COVID-19 Antigen is 98,7 %; 95% CI (97,40 % - 99,42%)

Please refer to the further information in the instructions for use.

MPG Healthcare

IMPORTER

Michael Pachleitner Group
Liebenauer Tangente 4
8041 Graz, Austria
info@mpg-healthcare.com

Date: 2021-03-15